



Concerned About Preparing Your Equipment For A Temporary Practice Closure?

It's not as easy as pulling the shades and locking the door...

Your Benco Service Team can help your offices to ensure systems will restart when you return! We did the homework by speaking with major manufacturers to get their recommendations:

- **WATER BOTTLE:** Remove, empty, replace; run system to purge water from unit lines; remove water bottle, let air dry
- **AIR COMPRESSOR:** Power down and release all air; drain tanks where applicable
- **VAC LINES:** Flush all vac lines; replace sediment filters and all disposable traps on units
- **AMALGAM SEPARATORS:** Replace amalgam separators due to prophy paste hardening
- **DRY VAC:** Power down and...
 - **Midmark Power and G Vac:** If cutting main power, add WD-40 to pump drive piston to prevent lockup. If leaving power on and newer style system will autorun every so often to prevent lockup.
 - **RamVacs:** Remove air filter to release pressure. Can hydro lock from oil drip? This is normal as motor overloads may trip on startup. Kill voltage to system.
 - **Vac Systems:** Clean out tanks where applicable
 - **Mohave Vacs:** Leave breakers on or MMC can risk serious errors
- **WET VACS:** Turn off water supply after flushing lines and turn off the power
- **AUTOCLAVES:** Clean and drain; do not replace water; let air dry
- **HYDRIM:** Remove soap/salt and solution and run shipping cycle
- **DISTILLERS:** Empty all distillers and let dry
- **WATER PURIFICATION SYSTEMS:** Shut down water and purge with air until operatories and all faucets run dry
- **CHAIRS:** Leave in upright position as high as they will go
- **HANDPIECES:** Oil all handpieces; clean and lube all HVE and SE handles
- **PLASTER TRAPS:** Remove and discard
- **COMPUTER SYSTEMS:** Complete backup on removable hard drive; take to safe location offsite; power down all computers, servers, switches, etc.
- **ULTRASONICS:** Drain and clean; let air dry
- **WATER COLLECTION CANISTERS:** Empty all on cassette sterilizers
- **MODEL TRIMMERS:** Run water for a period of 10 minutes with unit left on to clean wheel as well as flush all drain lines
- **DUST COLLECTOR FILTERS:** Replace all in lab areas
- **AIRLINES:** Bleed all airlines in operatories and lab areas
- **MILLS:** Thoroughly clean; drain coolant water
- **3D PRINTERS:** Remove all resin in machine and clean

IF YOU HAVE ANY QUESTIONS, PLEASE CALL BENCO'S SERVICE TEAM AT 1.800.GO.BENCO (1.800.462.3626) TODAY!

BencoDental™